

BARRIER CREAM

Stoma Auxiliary/ REF 6015



✓ Rejuvenate the skin texture



✓ Maintains Natural pH



✓ Easy to apply & absorbs quickly into the skin



✓ Protective shield for External aggressors



✓ Centella extract treats dry & sensitive skin



✓ Water repellent, retains skin's natural moisture

People with stomas face problems with their peristomal skin. It has been reported that up to 80% of such people have risk of sore peristomal skin*. It is essential to ensure healthy skin, free from breaks or soreness as irritated & damaged skin might lead to appliance leakage. Barrier Cream is used to treat, protect and aids in retaining the integrity of peristomal skin.

* [Peristomal skin care and the use of accessories to promote skin health - PubMed \(nih.gov\)](#)

BARRIER CREAM

Stoma Auxiliary/ REF 6015

- Non-sticky, Balm-like topical cream, makes a protective shield between the skin and external aggressors
- Highly moisturized, yogurt like texture is easy to spread and absorb quickly into the skin
- Ceramides hold the skin cells firmly together in the uppermost skin layer and creates a skin barrier for allergens .
- Skin friendly Cyclomethicone, helps in improving the skin texture and leaves a sensory feel.
- Antioxidant property of Centella extract complement the treatment of dry, macerated and sensitive skin
- Water Repellent formulation enables the skin to retain its natural moisture for longer.
- Consistent use of barrier cream rejuvenate the dull-looking skin and strengthen the cells to create a strong barrier.
- Available weight/tube: 60 g/ 2 Oz

Ingredients: Aqua, Light liquid Paraffin oil, Petroleum Jelly, Ozokerite wax, Glyceryl Oleate, Lanolin, Propylene Glycol, Glycerin, Magnesium Citrate, Cyclomethicone, Methyl Paraben, Propyl Paraben, Citric acid

Indications for use: For dry, sore and irritating skin, first degree skin abrasions, bruise, minor scrapes, cuts, tears, blisters or burns.

How to use: Clean the affected skin area followed by other skin care steps. Squeeze an adequate amount of cream onto affected skin area. Spread gently and massage with clean finger. Allow to stay it for 2 minutes. Wipe away the excess, if any, with a wet tissue.



KANAV HEALTHCARE PRIVATE LIMITED

308, Industrial Area Phase-1,

Panchkula-134113 Haryana (INIDA)

Customer Care: + 91 172 5027058, Mobile: 9910012114

Email: kanavhealthcare@gmail.com,

Website: www.kanavhealthcare.in



Lets Get Social

