

STOMA CARE STARTER KIT



Scan QR Code & Watch Video



Refer Stoma guide,
or Scan QR Code to
watch video



1. Measure the stoma size and Clean it



2. Select stoma device



3. Cut the stoma bag as per its size



4. Apply skin protection Film with applicator



5. Place stoma paste ring



6. Apply the stoma device



7. Fix the stoma Bag face plate & Belt



8. Cover the stoma bag



Use bath apron while bathing



STOMA CARE STARTER KIT

REF K 01 /K 02 /K 03 /K 04 /K 05 /K 06

✓ Compact kit designed with compassion to serve people with stoma to start as a beginner

Kit Consists of	One-piece (K 01)	One-piece (K 02)	Convex (K 03)	Convex (K 04)	Two-piece (K 05)	Two-piece (K 06)
For Stoma Type	Colostomy & Ileostomy	Urostomy	Colostomy & Ileostomy	Urostomy	Colostomy & Ileostomy	Urostomy
Skin Protective Film Applicator (Ref 60320)	5 Nos.	5 Nos.	5 Nos.	5 Nos.	5 Nos.	5 Nos.
Stoma Paste Protective Ring (Ref 6051)	5 Nos.	5 Nos.	5 Nos.	5 Nos.	5 Nos.	5 Nos.
Type of Stoma Device	One Piece type Ostomy Pouch (Ref 12075) ; 5 Nos.	One Piece type Urostomy Pouch (4016001) ; 5 Nos.	One Piece type Ostomy Pouch (Ref 12057) ; 5 Nos.	One Piece type Ostomy Pouch Ref 40857 ; 5 Nos.	Skin Barrier Flange 60 mm (Ref 3016) & Transparent Ostomy Pouch (Ref 2146) ; 5 Nos. each	Skin Barrier Flange 60 mm (Ref 3016) & Urostomy Pouch (Ref 4036) ; 5 Nos. each
Stoma Bag Face Plate (Ref 6770)	1 No.	1 No.	–	–	1 No.	1 No.
Stoma Belt (Ref 50100)	1 No.	1 No.	1 No.	1 No.	1 No.	1 No.
Curved Scissor	1 No.	1 No.	1 No.	1 No.	1 No.	1 No.
Stoma Bag Bath Apron	1 No.	1 No.	1 No.	1 No.	1 No.	1 No.
Stoma Pouch Cover	1 No.	1 No.	1 No.	1 No.	1 No.	1 No.
Stoma care Guide	1 No.	1 No.	1 No.	1 No.	1 No.	1 No.

Scan & Watch Video



Lets Get Social



KANAV HEALTHCARE PRIVATE LIMITED
308,Industrial Area Phase-1, Panchkula-134113 Haryana, (INDIA)
Customer Care: +91 172-5027058, Mobile: 9910012114

E-mail: kanavhealthcare@gmail.com

www.kanavhealthcare.in

