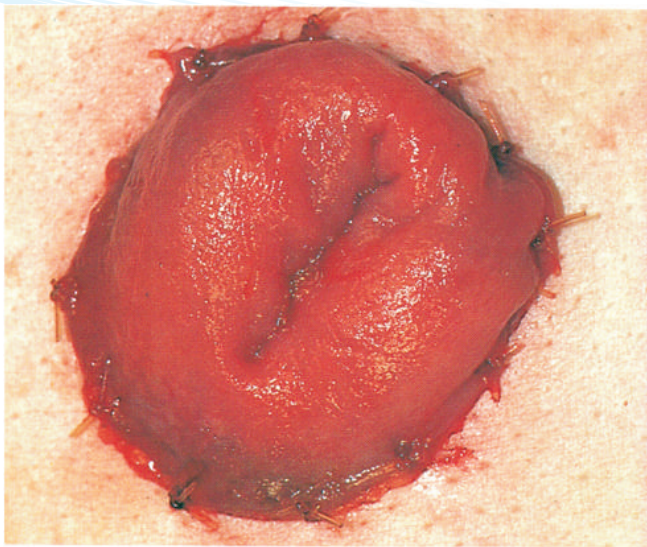
**KANAV**  
**Healthcare**

## STOMA CARE GUIDE

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## APPLICATION

Scan QR Code and watch online video OR ask for video on WhasApp@9910012114

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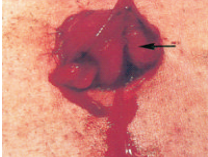
# TROUBLE SHOOTING

Leakage		keep pre-cut one piece pouches, tissues etc. in your hand bag while traveling or going outdoors. Use belt or abdominal binder for extra support.
Hernia		Ostomy Binder with opening for Ostomy pouch. All direction stretchable soft cotton fabric.
Peristomal Irritant Contact Dermatitis (PICD)		Contact Stoma Therapist. Minimise skin exposure to stomal output. Fill gaps between stoma and adhesive disc with paste.
Adhesive Trauma  Excoriation		Pull adhesive gently while removing. Use adhesive remover lotion or sanitizer solution for softening adhesive.
Skin Complications (Granuloma) Adhesive Trauma  Excoriation		Consult Stoma Therapist and Surgeon.
Stenosis/ Constipation/ Obstruction		Consult Stoma Therapist and Surgeon.
Prolapse		Consult Stoma Therapist and Surgeon.
BLEEDING		Consult Stoma Therapist and Surgeon. Apply Cold/ Ice Pack on Bleeding Area.

# GENERAL INSTRUCTION

**Odor & Ballooning :** Use bags with charcoal filter. Can use charcoal tablets or oxidizing solution or Ostomy Deodorant for reducing bad odor. Add in pouch after draining contents. Use pin puncture and sticker patch method for releasing excessive flatus / gas.

**Emptying of Pouch and Disposal :** Drain out the pouch in toilet / kidney tray when 1/3 is filled. Change pouch periodically. Wash pouch thoroughly & discard pouch in a plastic bag and dispose off in medical / hazardous waste.

## General Instructions :

### A) Trouble shooting for short wear time

Possible Cause	Management Option
Excessive weight	Empty frequently (usually when 1/3 is filled)
Wrong method of locking	Lock & unlock carefully. Follow manufacturer's guidelines.
Physical activity	Use Ostomy Binder / Belt during physical activity.
Climatic conditions	The adhesive strength may reduce at temperature above 40 c or below 10 c and also during high humidity & perspiration.
Leakage	Select right size of the appliance and avoid finger contact with adhesive surface during application.
Incorrect application	Follow correct methods of application.

### B) Back to living a normal life

- Clothing :** You can use the same clothes as before the surgery.
- Bathing :** With an ostomy, you can shower or take bath, just as you did before. If you are taking the bath along with the appliance, we suggest the use of plastic wrap over the appliance. You may also swim with the use of special mini cap, both in sea and in public swimming pools.
- Sexual Relationships:** People tend to believe that a colostomy will be a barrier to a normal sex life, but you should be aware not to create a false problem, You can enjoy just as natural sex life as before the surgery. Before sexual intercourse you may change to a smaller size of bag or use a special mini cap to cover stoma / Empty Pouch.
- Travel :** An ostomy needs not prevent you from travelling. But to be on safer side, you should carry extra bags and accessories for emergencies.
- Diet :**
  - Take small regular meals, preferably a balanced diet.
  - Gradually try to figure out what is ok and what to avoid in diet.
  - Prepare own diet chart or ask a dietician. Diet chart is available on [www.kanavhealthcare.in](http://www.kanavhealthcare.in)
  - Eat slowly and chew your food well with mouth closed.
  - Avoid chewing gum and use of straw while drinking.
  - Smoking and alcohol are injurious to health.

## ACCESSORIES

CODE	SIZE	DESCRIPTION
6012	STOMA PASTE	HYDROCOLLOID ADHESIVE PASTE FOR FILLING LEVELLING AROUND STOMA
6023	STOMA POWDER WITH ALOE VERA EXTRACT	OSTOMY POWDER WITH ALOE VERA EXTRACT
19	KARAYA POWDER	KARAYA POWDER FOR TREATING EXCORIATED / WEAPING / DAMAGES SKING
60320	FILM WIPE	SKIN PROTECTIVE FILM APPLICATOR
MS O6	OSTOMY BINDER	FOR EXTRA SUPPORT AND HERNIA PREVENTION
#03	OSTOMY BELT	SOFT BELT WITH CUSHIONING AND HOOKS FOR HOLDING OSTOMY POUCH
<b>HYDROCOLLOID SKIN BARRIER SHEETS</b>		
5000012	20 X 20 CM	HYDROCOLLOID SKIN BARRIER SHEET FOR SKIN PROTECTION AND EXCORIATED SKIN / FISTULA / WOUND DRAINAGE
5000014	25 X 30 CM	DO
5000015	10 X 10 CM	DO

# PRODUCT CODES

## ONE PIECE TYPE SYSTEM

CODE	SIZE	DESCRIPTION
1023 TNF	UPTO 60 MM	ONE PIECE TRANSPARENT POUCH WITH FILTER (SEPARATE CLAMP)
1117001F	UPTO 70 MM	ONE PIECE TRANSPARENT POUCH WITH FILTER AND FLEX CLAMP
1146001F	UPTO 60MM	ONE PIECE TRANSPARENT POUCH WITH FILTER AND VELCRO CLAMP
1146008F	UPTO 60MM	ONE PIECE OPAQUE POUCH WITH FILTER AND VELCRO CLAMP
10910201F	UPTO 102 MM	ONE PIECE TRANSPARENT POUCH WITH FILTER (SEPARATE CLAMP)
4016001	UPTO 60 MM	ONE PIECE TRANSPARENT POUCH FOR UROSTOMY WITH NON RETURN VALVE AND SPOUT
1155001F	UPTO 50 MM	ONE PIECE TRANSPARENT POUCH WITH FILTER AND VELCRO CLAMP FOR CHILDREN (PAEDIATRIC)
4025001	UPTO 50 MM	ONE PIECE TRANSPARENT POUCH FOR UROSTOMY WITH NON RETURN VALVE AND SPOUT FOR CHILDREN (PAEDIATRIC)
1213201F	UPTO 30 MM	INFANT POUCH - ONE PIECE

### CONVEX FLANGES & POUCHES

1143200FC	UPTO 32 MM	ONE PIECE CONVEX POUCH WITH VELCRO CLOSURE
1205700FC	UPTO 57 MM	ONE PIECE CONVEX POUCH WITH VELCRO CLOSURE
306320110C	UPTO 32 MM	TWO PIECE FLANGE COMPATIBLE WITH 60 MM POUCH
1143200FC	UPTO 32 MM	ONE PIECE CONVEX POUCH WITH VELCRO CLOSURE

### POST OP POUCH / WOUND DRAINAGE

12210204	UPTO 102 MM	POST OP POUCH / WOUND DRAINAGE SYSTEM, WITH WINDOW & SPOUT
----------	-------------	--

### TWO PIECE TYPE SYSTEM

3015001	UPTO 40 MM	HYDROCOLLOID SKIN BARRIER WITH FLANGE, 50 MM
2145008F	UPTO 40 MM	DRAINABLE OPAQUE POUCH WITH VELCRO AND FILTER MATCHING 50 MM FLANGE
2145001F	UPTO 40 MM	DRAINABLE TRANSPARENT POUCH WITH VELCRO AND FILTER MATCHING 60 MM FLANGE
4035001	UPTO 40 MM	UROSTOMY POUCH WITH NON RETURN VALVE AND SPOUT
3016001	UPTO 50 MM	HYDROCOLLOID SKIN BARRIER WITH FLANGE, 60 MM
3064501	UPTO 45 MM	HYDROCOLLOID SKIN BARRIER WITH FLOATING FLANGE, 60 MM
2146008F	UPTO 50 MM	DRAINABLE OPAQUE POUCH WITH VELCRO AND FILTER MATCHING 60 MM FLANGE
2146001F	UPTO 50 MM	DRAINABLE TRANSPARENT POUCH WITH VELCRO AND FILTER MATCHING 60 MM FLANGE
4036001	UPTO 50 MM	UROSTOMY POUCH WITH NON RETURN VALVE AND SPOUT MATCHING 60 MM FLANGE
3017001	UPTO 60 MM	HYDROCOLLOID SKIN BARRIER WITH FLANGE, 70 MM
2207008F	UPTO 60 MM	DRAINABLE OPAQUE POUCH WITH VELCRO AND FILTER MATCHING 70 MM FLANGE